

Spackman Cannula

General Information

The traditional reusable Spackman Cannula is liked by clinicians but gives great difficulty to Sterile Services who have to try and clean a long narrow cannula with a closed end.

To combat concerns over cleanliness SUS have produced single use versions of the instrument by matching the geometry of a reusable for equivalent uterine manipulation capability, but improving the clamp action.


SUS provide two choices of Spackman Cannula dependant upon the users requirements

- S029 – Fixed acorn Spackman Cannula
- S030 – Movable acorn Spackman Cannula

Both of these options are latex and DEHP free, provide a good seal on the cervix and are single use therefore reducing risk. The clamp locks itself into place and has two strong hooks for attaching the forceps.

How to Use

- 1 (S030 only) Sound uterus and set acorn to correct position, secure using screw
- 2 Insert Cannula
- 3 Apply tenaculum/volsellum, pull clamp back
- 4 Pull clamp back without touching the lever (self-locking) to ensure good seal
- 5 To release tenaculum/volsellum, press lever to slide clamp to distal end

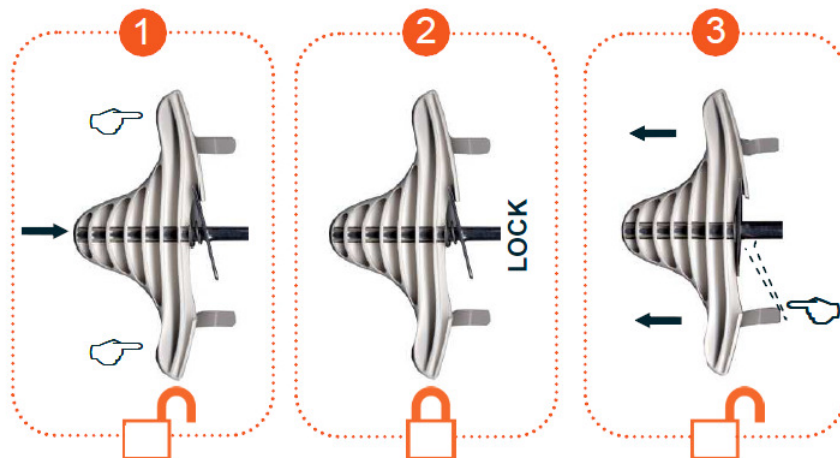

Movable Acorn- S030

Fixed Acorn – S029



Aequitas-Medical

Distributor Belgium: Aequitas-Medical
Kerkstraat 68 – 1601 Ruisbroek
Belgium
TEL +32 2 378 29 32
FAX +32 2 331 56 73
info@aequitas-medical.be / www.aequitas-medical.be




BBIC, Innovation Way, Bamsley
South Yorkshire S75 1JL UK
Telephone +44 (0)1226 732 333
Facsimile +44 (0)1226 785 386

Email info@susl.co.uk
Website www.susl.co.uk

General Surgery

Aspiration Needle
5 mm – 32 cm
S080

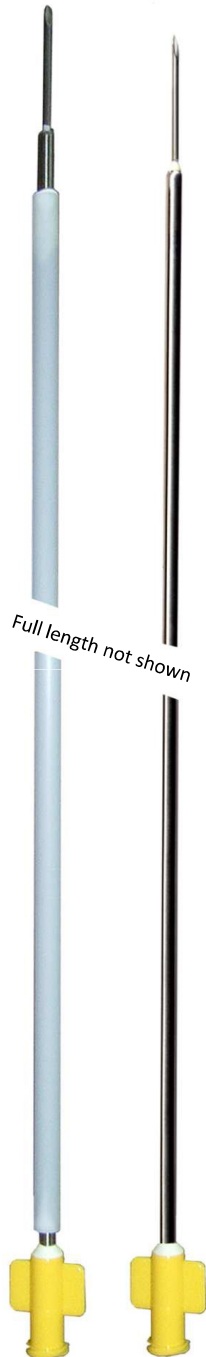

Aspiration Needle
3 mm – 32 cm
S019

Poole suction
S010



Aequitas-Medical

Distributor Belgium: Aequitas-Medical
Kerkstraat 68 – 1601 Ruisbroek
Belgium
TEL +32 2 378 29 32
FAX +32 2 331 56 73
info@aequitas-medical.be / www.aequitas-medical.be



BBIC, Innovation Way, Bamsley
South Yorkshire S75 1JL UK
Telephone +44 (0)1226 732 333
Facsimile +44 (0)1226 785 386

Email info@susl.co.uk
Website www.susl.co.uk